

# CATS® TONOMETER PRISM ACCURACY GUIDE

REVIEW PRIOR TO USE:



For improved IOP accuracy and mires brightness, set the slit lamp illumination to 45 degrees and start with the tonometer dial at 5mmHg.


## NOTE THE FOLLOWING:

- Unlike GAT, the CATS® Tonometer Prism will present distorted mires to alert users when the tip is not centered.
- After slit lamp adjustment, the CATS® Tonometer Prism mires will appear normal and round, ensuring accurate IOP measurement.

CENTERED



NOT CENTERED: DISTORTION

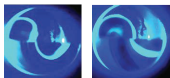

**FOR SUPPORT:** 1-520-975-4550 or visit [CATSIOP.COM/QuickStart](http://CATSIOP.COM/QuickStart)

SCAN HERE OR VISIT  
**[CATSIOP.COM/QuickStart](http://CATSIOP.COM/QuickStart)** TO  
VIEW THE ACCURACY /  
CENTRATION VIDEO

